

ARJOHUNTLEIGH

GETINGE GROUP

BATHING SOLUTIONS

FOR THE CENTRAL BATHING AREA



...with people in mind

CREATE THE OPTIMUM SYSTEM

ArjoHuntleigh bathing systems ensure an optimum environment to support high quality care and good working conditions throughout the assisted bathing cycle. We provide flexible solutions to meet safety and comfort needs in central bathing areas.

Meet changing needs

Our flexible systems can be adapted to meet the changing needs of patients and residents. The choice of bath lengths, bath shapes and multiple options means an optimum system can be tailored to suit current and future requirements of different facilities.

Enhance bathing benefits

Maximize the beneficial effects of bathing for mind and body. Our options enhance the bathing experience with the deep relaxing effects of *Hydromassage™* and the sensitive cleaning action of *Hydrosound®*.

Secure disinfection and hygiene

Having an effective way to combat the ever-present risk of cross-infection and harmful microorganisms is vitally important. Our bathing systems offer easy-to-clean surfaces and an effective, well-proven integrated disinfection system.

Safety and quality

In addition, these bathing systems are manufactured to Good Manufacturing Practice (GMP) guidelines and meet UL2601-1, CAN/CSA C22.2 No 601.1-M90 electrical and relevant UL and ANSI plumbing standards. For optimum system performance, servicing is carried out by our service technicians.



HYGIENE LIFTS

Hygiene lifts are an integral part of the bathing system, providing safe and comfortable support for the resident/patient during bathing, lifting, transfers and transport. ArjoHuntleigh offers four hygiene lifts to suit different needs and requirements.

A complete bathing system consists of a height-adjustable bath and an integrated height-adjustable mobile hygiene lift. The height-adjustable elements of the system ensure good ergonomics, as the caregiver can always adjust equipment to a comfortable working height.

It is vital for the resident's/patient's safety and mobility that hygiene lifts provide the right balance of support and mobilization. Our four hygiene lifts – the *Alenti*[®] and *Calypto*[®] lift hygiene chairs and the *Miranti*[™] and *Bolero*[®] lift bath trolleys are designed for specific mobility levels. These hygiene lifts cover all levels in the ArjoHuntleigh *Mobility Gallery* from Albert to Emma.

The powered functions of our lifts reduce strain on caregivers. With powered raising and lowering of the resident/patient, manual lifting and transfers are eliminated from the bathing cycle. Our powered hygiene lifts are very easy to use, enabling a single caregiver to safely manage the entire bathing cycle.

Once on the seat or stretcher of the hygiene lift, the resident/patient remains there in comfort and safety throughout the bathing cycle – from bed to bath and back again – for transfers, transport and immersion in the bath itself. The resident/patient can therefore enjoy the full benefits of bathing, optimally supported by the lift at all stages.

A simple 8-step selection process pinpoints your optimum bathing system:

1. Mobile hygiene lifter

The hygiene lifter is an integral part of a bathing system. Our *Mobility Gallery*[™] (see back cover) helps you select a lift to match the functional mobility levels of your residents.

2. Bath length

The resident's/patient's functional mobility level and the required hygiene lift are decisive factors in choosing the right bath length – short, standard or extra-long.

3. Bath shape

We offer a straight bath or a keyhole-shaped tub, designed specifically for smooth integration with mobile hygiene lifts.

4. Options to enhance bathing benefits

Incorporating *Hydromassage*[™] or *Hydrosound*[®] in your system can enhance the benefits of bathing for your residents/patients.

5. Safety and protection

The control panels at the foot-end of the tub provides different levels of control to manage filling routines and water temperature.

6. Disinfection

There is a more effective alternative to manually cleaning and disinfecting the bath. A built-in disinfection system provides maximum protection against cross infection. The control panel offers automatic cleaning and disinfection of all *Hydromassage* components at the touch of a button.

7. Bath liquids

A range of bath oils and a shampoo & body wash from ArjoHuntleigh are designed to support gentle cleaning routines and promote good skin care.

8. Accessories

Enhance the physical comfort and convenience of bathing sessions with accessories to meet your resident's/patient's needs.

RHAPSODY & PRIMO



Rhapsody & Primo

These two baths ensure bathing sessions with the highest level of safety and comfort, and are easily adaptable to meet the needs of different care facilities.

Rhapsody[®] and *Primo*[™] baths from ArjoHuntleigh are not only height-adjustable, but also highly adaptable. Our bathing systems are designed for flexibility and are compatible with a range of transport and lifting aids that cover all resident mobility levels. A complete system enables one caregiver to safely manage the entire bathing cycle. Bath features and options can be combined to create systems that suit the specific needs of your facility's residents/patients.

With our easy-to-follow selection guide on the next pages, you can choose from the alternatives for bath shape, length and treatment options, including either *Hydromassage* or *Hydrosound*.



The bathing system includes two bath shapes, a traditional straight-edged bath, *Primo* (shown above), or a more innovative keyhole-shaped *Rhapsody*. This shape enhances patient comfort with optimum immersion and extra room for shoulders and arms.

Hydrosound Advanced Cleaning Technology

Hydrosound offers a gentle touch for cleaning sensitive skin. This advanced system helps to protect skin by offering a pleasant and relaxing alternative to manual washing. An integrated ultrasound unit in the bath generates millions of microscopic bubbles which gently vibrate against the skin. The imploding bubbles remove dirt particles from the skin in a gentle brushing action.



Standard Rhapsody/Primo Features:

All systems include:

- Hydraulically height adjustable
- Digital temperature display for water temperature
- Integrated liquids dispensing system
- Integrated cleaning and disinfection system
- Lockable liquids storage compartment
- Pressure and Temperature Balanced Mixing valves for maximum anti-scald protection
- Autofill standard
- Built-in shower system with trigger handle shower design
- Built-in adjustable foot support and head pillow come standard

RHAPSODY & PRIMO


Panel



P 300


- All standard features
- Programmable temperature control with motorized mixing valve
- Automatic disinfection system with contact time lock out
- Audible and visual warning alarm system if disinfectant container is empty
- Emergency lowering
- Three level programmable autofill

Which bath shape?

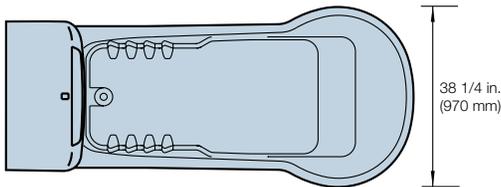

Rhapsody


The innovative key-hole shaped bath has superior resident/patient comfort when integrating a lift hygiene chair. Key-hole shape with integrated arm rests allow ultimate comfort for your residents/patients with more width than standard shaped baths.

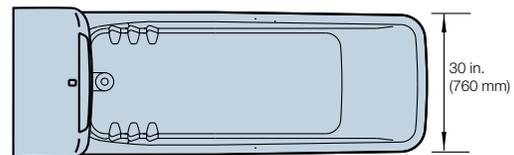

Primo

Primo design is a traditional shaped bath design which offers ample room for you residents/patients, but offers a smaller footprint when space is limited.


Which bath length?

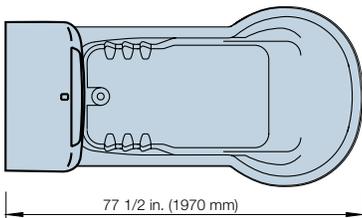

Short bath (System 20)

Rhapsody and Primo


- When space cannot fit the standard length or extra long bath
- This option will work only with *Alenti* and *Calypso* hygiene chairs
- User range: mobility levels A-B-C in *Mobility Gallery*

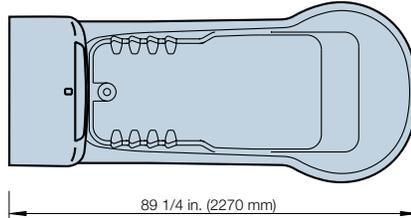

Standard length bath (System 23)

Rhapsody and Primo

- Offers excellent immersion
- Integrates with all hygiene lifts
- Best choice for A-B-C-D-E in *Mobility Gallery*

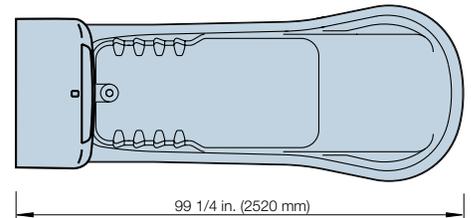

Extra-long bath (System 25)

Rhapsody

- Offers optimum immersion
- Integrates with all hygiene lifts
- Ideal for larger residents/patients
- Ideal for severely contracted or rigid individuals

ACCESSORIES



ArjoHuntleigh bathing systems offer a wide range of accessories to improve key aspects of the bathing session – from enhancing individual patient comfort and safety to making bathing a more pleasurable experience.



Integrated liquid dispensing system

The bathing system offers a convenient way to dispense shampoos, body washes and tub disinfectants.



Pillow

The pillow is attached to the tub by suction cups. Once positioned for personal comfort, the pillow remains firmly in place, but is easy to reposition when required.



Foot support

The movable foot support can be slotted into one of four positions and provides extra security for residents/patient by preventing them from slipping down in the tub.



Snap-on toiletry tray

A snap-on toiletry tray allows shampoo, bath oils, toiletries and other personal items to be kept within easy reach.



Treatment hose

The treatment hose connects directly to the *Hydromassage* system, making it possible to focus *Hydromassage* for localized massage on an arm or leg.

CHOOSE THE RIGHT HYGIENE LIFT

1. Mobility is the starting point

The functional mobility of your residents/patients is the determining factor in the choice of a hygiene lifter. Our *Mobility Gallery* is a tool to help you identify specific mobility levels.

2. Which bath length?

All lengths of the two baths are compatible with our lift hygiene chairs. The lift bath trolley works with standard length *Rhapsody/Primo* tubs and the extra-long *Rhapsody*.

3. Lift hygiene chair or lift bath trolley?

Our lift hygiene chair serves less dependent residents/patients, specifically Albert, Barbara and Carl, while our lift bath trolley offers optimum support for Doris and Emma.



Keep going with 2 x power

All units are delivered with two high-capacity batteries. With one battery always fully charged, ArjoHuntleigh hygiene lifts offer maximum availability, as they are always ready to use, 24 hours a day.



Handset control

Efficient control of powered functions such as raising, lowering and central braking via the handset allows the caregiver to work close to the resident/patient at all times, which improves personal contact and quality of care.



LIFT HYGIENE CHAIR - ALENTI



Bedside transfers

Transferring the resident/patient from the bed is an efficient and safe routine. The resident/patient is transferred onto the *Alenti* seat.



Transfer and transport

After the transfer, the seat is raised to the optimum height for transport.

Alenti - the most advanced lift hygiene chair

***Alenti*[®] is an integral part of a safe, comfortable bathing cycle. This lift hygiene chair from ArjoHuntleigh gives the resident/patient superior comfort and constant security during transfers, transport and the bathing session itself.**

Alenti is the most comfortable and best available lift hygiene chair to cover the needs of semi-dependent residents. With *Alenti*, a single caregiver can safely manage all aspects of the bathing cycle and work in an ergonomically sound way at all times.

The battery-powered *Alenti* offers many features designed to enhance safe ergonomic working practices – from handset control and height-adjustability to ergo grips and central braking. *Alenti* can also be used as a shower, toileting, weighing and general transport chair.

Using the height-adjustable *Alenti* in conjunction with a height-adjustable ArjoHuntleigh bath is the perfect combination for optimized ergonomics, as caregivers can always perform routines at the correct working height.



Powered functions Dual Controls for:
raising, lowering and electronic central braking

Bath compatibility:
all lengths of *Rhapsody* and *Primo*

Best suited for:
mobility levels Albert, Barbara and Carl

Product Information	
Lifting capacity	400 lbs (182 kg)
Weight	143 lbs (65 kg)
Total width (Seat + Chassis)	23 ¾" (605 mm)
Total length	35 ½" (900 mm)
Seat height	18 ¾" (475 mm) - 41 ¾" (1060 mm)
Lifting stroke range	23" (585 mm)



Integrated scale

Choose the on-board electronic scale option and *Alenti* becomes a convenient alternative to fixed scales for weighing residents/patients.



Safety belt

The comfortable belt ensures optimum support, so the resident/patient stays securely in place during bathing and transport.



Bedpan

Alenti can also be used for toileting. Extra security for incontinent residents/patients can be provided with the bedpan & bedpan holder option.



Complete control

In addition to the separate mast mounted controls; the raising, lowering and central braking functions are efficiently controlled with a handset, enabling the caregiver to stay close to the resident/patient.

LIFT BATH TROLLEY - MIRANTI



Bedside transfer

The side-positioned pillar enables efficient sideways transfer from the bed. Adjust the height of stretcher to rest on the bed, then roll the resident/patient gently onto it.



Transfer and transport

With the *Miranti*, one caregiver can transfer the resident/patient at the bed and take them directly to the bath.

Miranti - the most advanced lift bath trolley

The *Miranti*[™] lift bath trolley offers an unrivalled level of power-assisted care in the lifting, transporting and bathing of more dependent and totally dependent residents/patients. With Power Drive (optional) and adjustable powered backrest, *Miranti* sets new standards for ergonomic, labor-saving operation and resident/patient comfort and safety.

The *Miranti* makes bathing a dependent resident/patient a one-to-one process. For the resident/patient, being bathed by a single caregiver is less stressful and intrusive than team care. Powerful support functions, ergonomic design, easy operation and simple comfort and safety features create system confidence for both the caregiver and resident/patient. The caregiver has a better working environment and the resident can relax, enjoy being in the bath, and gain the full benefits of bathing.

The *Miranti* is built for intensive use. The rechargeable battery system means it can operate around the clock. Time savings allows more residents to be bathed with one or two caregivers, increasing the amount of remaining staff to do other jobs.

The efficient functions and features of the *Miranti* give the caregiver more time to attend to the resident/patient and let them enjoy the full benefits of the bath.



Powered functions:

raising, lowering and electronic central braking, adjustable backrest and Power Drive

Bath compatibility:

with *Rhapsody* and *Primo* (standard length) and *Rhapsody* (extra-long)

Best suited for:

mobility levels Doris and Emma

Product Information	
Lifting capacity	350 lbs (160 kg)
Weight	103 lbs (227 kg)
Total width (Seat + Chassis)	37" (940 mm)
Total length	76 ¾" (1950 mm)
Seat height	17 ¾" (450 mm) - 40 ¾" (1035 mm)
Lifting stroke range	23" (585 mm)



Thigh support and head cushion

The thigh support cushion and multi-slot head cushion can be adjusted to different positions for individual comfort.



Safety belt

When extra security is needed, two safety belts are available to ensure the resident is safely positioned and secure at all times during bathing and transport.



Integrated scale

Weighing residents on the *Miranti* at bedside or during transport is simple using the optional on-board electronic scale.




Power Drive (optional)

Controlled via a waterproof push-button in the steering transport handle, the caregiver can easily transport the resident/patient to the bath with minimal effort.

PRODUCT SPECIFICATIONS

Space Requirement

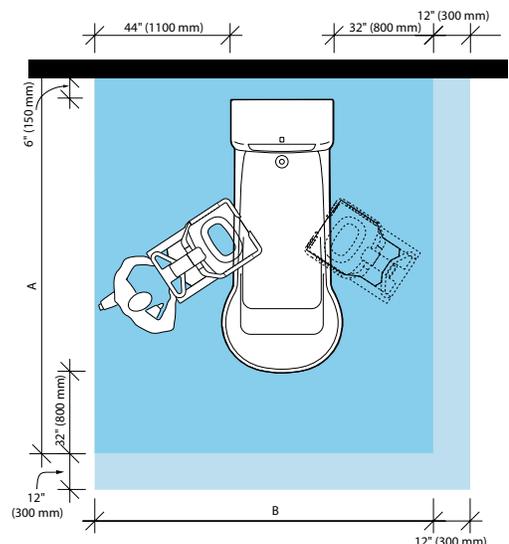

Short	Standard	Long	Primo S & L*
A=114" (2895 mm)	A=125 3/4" (3195 mm)	A=135 1/2" (3445 mm)	A=114", 125 3/4" (2895, 3195 mm)

Blue area shows the minimum working area required for the staff to be able to use the mechanical aids in an ergonomic way from one side.

Light blue area shows required extension of working area to facilitate activities from either side to provide adequate access for the resident, mechanical aid and assisting caregiver.

Mobility Gallery



- A Albert**, who is ambulatory and independent
- B Barbara**, who can support herself to some degree
- C Carl**, who sits in a wheelchair and has little capacity to support himself
- D Doris**, who has no capacity to support herself
- E Emma**, who is almost completely bedridden and totally dependant

Please contact ArjoHuntleigh for further information on the *Mobility Gallery*™.

Product Information

Rhapsody Short (System 20)	Standard (System 23)	Long (System 25)	Primo Short & Standard
Total Weight			
860 lbs (390 kg)	1036 lbs (470 kg)	1157 lbs (525 kg)	838 lbs (380 kg)
Water Consumption*			
49 gal (185 Liters)	69 gal (260 Liters)	82 gal (310 Liters)	48, 66 gal (180, 250 Liters)
Filling Time at 300 kPa dyn. pressure (Hw 80° C)*, min.			
3.00 min.	4.50 min.	5.15 min.	3, 4.30 min.
Emptying Time, min.			
3.00 min.	4.50 min.	5.25 min.	3, 4.30 min.
Min. Height from tub rim			
33" (835 mm)	33" (835 mm)	33" (835 mm)	33" (835 mm)
Max Height from tub rim			
52 1/4" (1330 mm)	52 1/4" (1330 mm)	52 1/4" (1330 mm)	52 1/4" (1330 mm)
Total Length			
77 1/2" (1970 mm)	89 1/2" (2270 mm)	99 1/4" (2520 mm)	77 5/8", 89 1/2" (1979, 2270 mm)
Total Width			
38 1/4" (970 mm)	38 1/4" (970 mm)	38 1/4" (970 mm)	33 1/2" (850 mm)
Total Weight			
860 lb (390 kg)	1014 lb (460 kg)	1158 lb (525 kg)	838, 1014 lb (380, 460 kg)
Total Water Consumption*			
49 gal (185 Liters)	68.5 gal (260 Liters)	82 gal (310 Liters)	39.5, 55 gal (180, 250 Liters)

* Completely filled with water and resident

Use only ArjoHuntleigh designed parts, i.e. parts specifically designed for the purpose, on equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development, we reserve the right to modify designs and specifications without prior notice. ® and ™ are trademarks belonging to the ArjoHuntleigh group of companies. © ArjoHuntleigh, 2012

ARJOHUNTLEIGH

GETINGE GROUP

www.ArjoHuntleigh.com

ArjoHuntleigh Inc.
2349 West Lake Street
Addison, Illinois 60101
Phone: 800-323-1245
Fax: 630-925-7969

ArjoHuntleigh Canada Inc.
1575 South Gateway Road Unit C
Mississauga, Ontario L4W 5J1
CANADA
Phone: 800-665-4831
Info.Canada@ArjoHuntleigh.com



Use only ArjoHuntleigh designed parts, i.e. parts specifically designed for the purpose, on equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development, we reserve the right to modify designs and specifications without prior notice. ® and ™ are trademarks belonging to the ArjoHuntleigh group of companies. © ArjoHuntleigh, 2012