ARJOHUNTLEIGH

GETINGE GROUP


MINUET 2

COMMUNITY BED RANGE



...with people in mind

PRO-CONTOUR® ADVANCE PROFILING

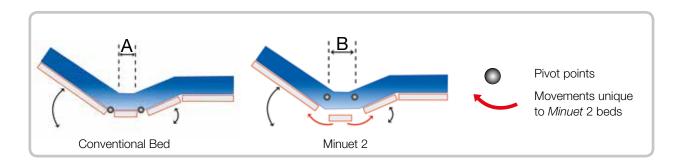

Compared to a conventional profiling bed (A), the *Pro-Contour Advance Profiling* on the four-section *Minuet 2* beds raises the pivot point of the backrest and thigh section closer to the client's natural pivot point (their hip joint) (B). As a client moves from a lying to a more upright position, this reduces abdominal compression, and improves comfort.

This offers major benefits over simple profiling beds:

- Excellent postural support
- Exceptional comfort

Intelligent profiling system



Integral bed extension



On descent from a profiled position the knee-break stays raised until the backrest reaches the same angle (A).

Both the backrest and knee-break then descend together until they reach a flat position simultaneously (B).

This intelligent profiling system maximises comfort, improves postural support and helps to reduce sliding down the bed. This has an associated benefit of reducing pressure, shear and friction. This can help to reduce the risk of pressure ulcers, and also moving and handling risks for carers associated with repositioning of the client.

To cater for taller clients, *Minuet 2* has an integrated mattress platform extension of 130mm.

This is easily engaged and offers maximum flexibility. Full profiling of the calf section is achieved even when the bed is extended, enhancing comfort for the client.






THE NEW BED STANDARD

Developed by the International Electrotechnical Commisssion (IEC), the IEC60601-2-52:2009 Standard is the new benchmark for medical bed performance and safety. Whilst the standard covers many aspects including electrical safety, load bearing, stability and durability, one of the main requirements focuses on reducing risk of patient entrapment in and around the bed frame.

Use of extensive incident and risk data has enabled the IEC to identify a number of areas around the bed where entrapment of a patient's head, neck or chest might result in a serious or life-threatening situation. By prescribing stringent test parameters for these areas, the aim is to reduce entrapment risks to the minimum possible level during operation and use of the bed.

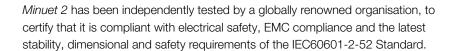



Maximising patient safety







These additional stickers (added to the bed frame) add valuable advice on recommended mattress size, patient height and ensuring the correct safety sides are added for maximum patient safety.

EASY TO CLEAN, TRANSPORT AND ASSEMBLE









A REAL SOLUTION FOR INDEPENDENT LIVING

The *Minuet 2* bed range takes many of the features and benefits of our acclaimed acute care range and delivers them in a package optimised for use in the residential setting. Advanced electrical profiling, low platform height and easy transport and storage combine with an attractive domestic appearance, to make *Minuet 2* the natural choice for the private, residential and convalescent home user.

- Low platform height 250 mm (10") from the floor
- Safe Working Load 180 kg (396 lbs)
- Maximum patient weight 145kg (320lbs)
- Electric backrest, knee-break and height adjustment
- Pro-Contour® Advance profiling associated with reduced pressure, shear and friction
- Battery back-up
- Integrated mattress platform extension of 130 mm (5") with mattress retainer
- Designed for use in private, residential and convalescent homes
- Easy to disassemble into four parts for improved installation access
- Optional two-way tilt; foot-down/head-down*
- Compact storage/transportation (using transit brackets)
- Compatible with a wide range of pressure-re-distributing mattresses from ArjoHuntleigh
- Wide range of accessories available

FEATURES (160 & 161)

- Electric height and backrest adjustment
- Electric knee-break adjustment
- Battery back-up
- Handset with raised, embossed buttons
- Lockable handset as standard
- Rising, retracting backrest
- Pro-Contour® Advance profiling
- Auto-Contour button
- Ratchet calf section adjustment
- Integral bed extension with mattress retained
- Beech effect head and foot end panels
- Removable head and foot end panels
- Slatted mattress platform
- 80mm (31/8") diameter castors with 'kick-stop' brakes
- Disassembles into 4 sections for transport/installation and storage

EXCLUSIVE TO MODEL 161:

• Electric foot-down/head-down tilt

IMPORTANT:

This model is NOT to be used within the domestic homecare environmen



^{*} Available with model 161



Full length wooden safety sides (pair) **CM-ACC00.52**



Full length steel safety sides (pair) CM-ACC04.52



Steel folding safety sides (pair) CM-ACC06.52



Steel extra height folding safety sides (pair) **CM-ACC07.52**



Product Code Description CM-ACC00-1.52 Full Length Standard Height Wooden Safety Sides (Grey/Beech) CM-ACC00-3.52 Full Length Standard Height Wooden Safety Sides (Brown/Beech) CM-ACC01-1.52 Full Length Standard Height Wooden Safety Sides (Grey/Cherry) CM-ACC00-1.14 Full Length Extra Height Wooden Safety Sides (Grey/Beech) CM-ACC00-3.14 Full Length Extra Height Wooden Safety Sides (Brown/Beech) CM-ACC01-1.14 Full Length Extra Height Wooden Safety Sides (Grey/Cherry) CM-ACC01-3.14 Full Length Extra Height Wooden Safety Sides (Brown/Cherry) CM-ACC03 Pads for Full Length Safety Sides CM-ACC04-1.52 Full Length Standard Height Steel Safety Sides CM-ACC06-1.52 Steel 3/4 Length Standard Height Folding Safety Sides CM-ACC07-1.52 Steel Extra Height Folding Safety Sides CM-ACC09 Lifting Pole with strap and handle CM-ACC11 Mobility Support Handle (to fit left side of bed) CM-ACC12 Mobility Support Handle (to fit right side of bed) CM-ACC13 Flexible Handset Holder CM-ACC14 IV Pole CM-ACC15 Mattress Pump Bracket CM-ACC17 Urine Bag Holder CM-ACC22-1.52 Egress Assist Rail Standard Height (Grey) CM-ACC23-1.52 Egress Assist Rail Extra Height (Grey) CM-ACC24 Mattress Retainers (supplied with the bed) CM-ACC26 Full Length Safety Side pads with top cushion



Mobility support handle

CM – ACC11 Left handed

CM – ACC12 Right handed



Flexible handset holder CM – ACC13



Lifting pole **CM - ACC09** Single position



Pump bracket CM - ACC15

PRODUCT SPECIFICATIONS

160 & 161	
Overall length	2250 mm (88,6")
Overall length (extended)	2390 mm (94")
Overall width without safety sides	940 mm (37")
Overall width with safety sides	1050 mm (41,3")
Mattress size (length x width x height)	1980 x 860 X 125 mm (78 x 33,8 x 5")
Height range	250 - 660mm (9.8 - 26")
Tilt range (if fitted)	+/- 10.8°
Backrest angle (maximum)	69°
Thigh elevation (maximum)	32°
Safe working load	180 kg (396 lbs)
Maximum patient weight	145kg (319lbs)
Section weights	
Head end	22.8 kg (50 lbs)
Foot end	22.8 kg (50 lbs)
Foot end mattress platform section	19.8 kg (44 lbs)
Head end mattress platform section	23.5 kg (52 lbs)
Electrical data (all models)	
Power in	1.7A max. 120V a.c. 60Hz or 1A max. 230 V a.c. 50 Hz
Electric shock protection	Class II, Type B
Ingress protection	IP24
Battery	12V, 1.2Ah sealed rechargeable lead/acid gel

Only ArjoHuntleigh designed parts, which are designed specifically for the purpose, should be used on the equipment and products supplied by ArjoHuntleigh. As our policy is one of continuous development we reserve the right to modify designs and specifications without prior notice.

 $\ensuremath{\mathbb{R}}$ and $\ensuremath{^{\text{TM}}}$ are trademarks belonging to the ArjoHuntleigh group of companies.

© ArjoHuntleigh, 2014







ARJOHUNTLEIGH GETINGE GROUP

www.ArjoHuntleigh.com

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of **ArjoHuntleigh**, **GETINGE** and **MAQUET**. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.



ArjoHuntleigh AB

Hans Michelsensgatan 10 211 20 Malmö Sweden

Phone: +46 (0) 10 335 45 00

ArjoHuntleigh International Ltd

Houghton Hall Business Park Houghton Regis

Bedfordshire, LU5 5XF, United Kingdom

Phone: +44 1582 745 800 Fax: +44 1582 745 866

E-mail: international@arjohuntleigh.com